

B!FLEX


PRELOADED



3 CLICKS FOR EASY,
SAFE AND PREDICTABLE
INJECTION


MEDJET PIL-MA

Single use injection system for Bi-Flex preloaded monofocal, toric and diffractive progressive hydrophilic IOLs

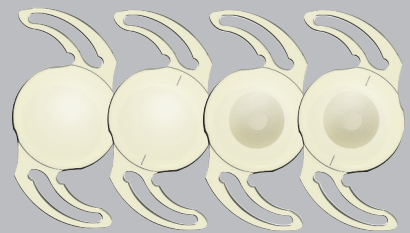


1st CLICK LOADING

Insert the injector to the wet container

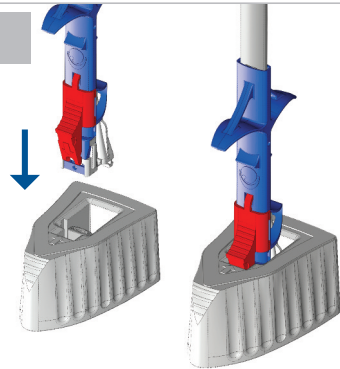


2nd CLICK CLOSING

Fold up the cartridge nozzle

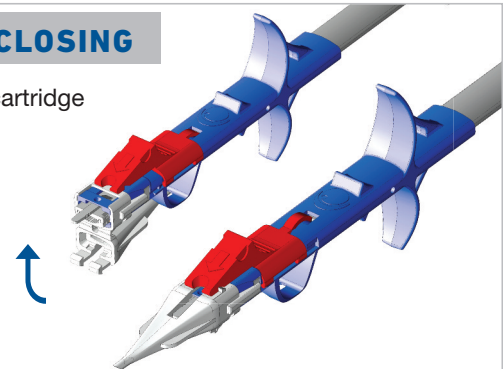


3rd CLICK READY

Push the red stopper forward to the end position and fill visco

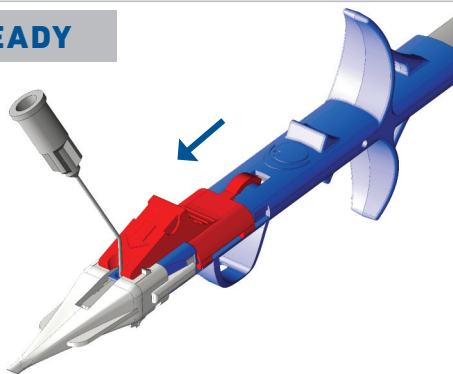

INJECTION

Push the plunger slowly to the end position

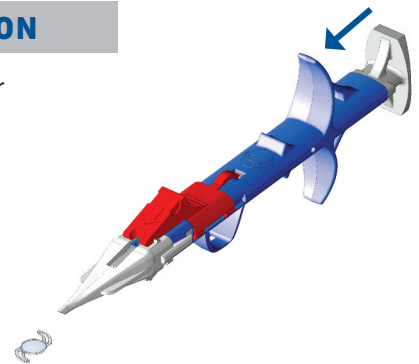

Lens types	Single-piece aspheric IOLs, clear and yellow • Monofocal • Toric • Diffractive Progressive • Toric Diffractive Progressive (Patent pending)
-------------------	--

Diameters	6.0 mm (optic) 13.0 mm (overall)
------------------	-------------------------------------

Angulation	0° – asymmetrical design
-------------------	--------------------------

Material	Copolymer of hydrophobic and hydrophilic monomers 25% water content with UV Absorber / + blue light filter (390–470 nm)
-----------------	--

Incision size	2.2 mm
----------------------	--------

PCO protection	360° Special Square Edge
-----------------------	--------------------------

A-Constant	118.9 (SRK/T)
-------------------	---------------



677P/677PY

Powers available -10.0 D → -1.0 D · (1.0 D steps)
0.0 D → +30.0 D · (0.5 D steps)



677PT/677PTY

Powers available -10.0 D → +30.0 D · (0.5 D steps)
Cylinders available 1.00 D, 1.50 D, 2.25 D, 3.00 D, 3.75 D, 4.50 D



677PMY

Powers available 0.0 D → +30.0 D (0.5 D steps)
Addition +3.5 D



677PMTY

Powers available +10.0 D → +30.0 D (0.5 D steps)
Cylinders available 1.00 D, 1.50 D, 2.25 D, 3.00 D, 3.75 D, 4.50 D
Addition +3.5 D

LA-IMCMIJPLMAENG 05 201506